## **NEW!**





# **Blood Preg Test**

### The name of the test is BioPRYN®

It is a pregnancy test that is done in the lab on a blood sample from a cow or heifer.



#### Is it accurate?

YES - 97% accurate. Open cows just aren't missed. False pregnancies (false positives) are rare but can occur if the embryo is dead or dying when the sample is taken — so you would re-test.

#### How do I use it?







- Best used in consultation with herd veterinarian
- Embryo safe
- Take blood sample at least 30 days after breeding and at least 90 days after calving
- Collect 2 mL of blood in a red-top tube and label with cow ID
- Fill out lab submission form
- Package for safe delivery (no ice needed) and ship to Palliser Labs (Lethbridge)

(Palliser Labs also offers milk cultures and disease testing for Johne's, BVD, Leukosis, Brucellosis and Neospora.)

#### BioCheck

c/o Palliser Animal Health Laboratories Ltd. #6, 320 W.T. Hill Blvd. South Lethbridge, AB T<sub>1</sub>J<sub>4</sub>W<sub>9</sub>

Phone: (403) 328-1844 Fax: (403) 328-1798

Email: pahlab@shawbiz.ca