

# IDEXX Milk Pregnancy Test

Test With Confidence™

## Pregnant? Confirm it with milk.



### Optimize reproductive efficiency with the IDEXX Milk Pregnancy Test.

Now you can offer your customers even greater value using existing milk samples. The IDEXX Milk Pregnancy Test is the simplest, most cost-effective way to confirm open cows and maximize overall herd health management programs.

- **Accurate determination of pregnancy status.** The IDEXX Milk Pregnancy Test delivers a high level of sensitivity and specificity from  $\geq 35$  days postbreeding and day 60 postcalving.
- **Trusted IDEXX ELISA technology.** Seamlessly integrates into existing lab-based work flows, allowing you to offer greater value to your customers at little impact to your operation.
- **Timely results.** Uses proven IDEXX ELISA technology for trusted results in less than 3.5 hours.
- **Cost-effective and hassle free** way to maximize overall herd health management programs.

Learn how to maximize the value of milk samples and optimize your business today.

# The IDEXX Milk Pregnancy Test

## Give your clients greater value with accurate, easy confirmation

- Boost client confidence—and your reputation—with definitive confirmation of pregnancy status.
- Demonstrate that you're a trusted partner in reproductive health by helping clients improve reproductive performance and shorten calving intervals.
- Run more pregnancy tests because clients trust your results, value the quick turnaround and rely on you for confirmatory testing.
- Expand your testing options, offering customers a comprehensive herd health management program using existing samples.

## Performance on par with ultrasound/palpation

		Ultrasound/Palpation	
		Pregnant (n=923)	Open (n=392)
IDEXX Milk Pregnancy Test	Pregnant	872	10
	Recheck	40	9
	Open	11	373

Sensitivity: 98.8% (95% CI = 97.7-99.3%)\*

Specificity: 97.4% (95% CI = 95.2-98.6%)\*

Rechecks: 4% of total tested (3% pregnant & 1% open)

\*Performance of the IDEXX Milk Pregnancy Test was determined by comparing test results to those from ultrasound/palpation (N = 1315 cows.) Rechecks were not included in performance calculations. See IDEXX Milk Pregnancy Test validation report for full test performance data.<sup>1</sup>

For more information about the **IDEXX Milk Pregnancy Test**, please contact your IDEXX representative or visit [idexx.com/reproduction](http://idexx.com/reproduction).

### Corporate Headquarters IDEXX Laboratories, Inc.

One IDEXX Drive  
Westbrook, Maine 04092  
USA

Tel: +1 207 556 4890 or  
+1 800 548 9997  
Fax: +1 207 556 4826 or  
+1 800 328 5461

### European Headquarters IDEXX Europe B.V.

Scorpius 60 Building F  
2132 LR Hoofddorp  
The Netherlands

Tel: +31 23 558 70 00 or  
+800 727 43399  
Fax: +31 23 558 72 33

### Asian Headquarters IDEXX Laboratories, Inc.

3F-5 No. 88, Rei Hu Street  
Nei Hu District  
11494 Taipei  
Taiwan

Tel: +886 2 6603 9728  
Fax: +886 2 2658 8242

#### Reference

1. Data on file at IDEXX Laboratories, Inc. Westbrook, ME USA.

© 2012 IDEXX Laboratories, Inc. All rights reserved. • 09-71749-00-EN-A

IDEXX and Test With Confidence are trademarks or registered trademarks of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries. The IDEXX Privacy Policy is available at [idexx.com](http://idexx.com).